TOXIC SUBSTANCE CONTROL ACT (TSCA) CERTIFICATION FORM INSTRUCTIONS

TOXIC SUBSTANCE CONTROL ACT	<u>r (TSCA)</u>	
CERTIFICATION		
DATE: November 1, 2017 (CHECK ONE SECTION ONLY) POSITIVE CERTIFICATION: "I CERTIFY THAT ALL CHEMICAL SU THIS SHIPMENT COMPLY WITH ALL RULES OR ORDERS UNDER TSCA AND OFFERING A CHEMICAL SUBSTANCI VIOLATION OF TSCA OR ANY APPLI ORDER THEREUNDER."	D THAT I AM NOT F FOR ENTRY IN	
NEGATIVE CERTIFICATION: X I CERTIFY THAT ALL CHEMICALS ARE NOT SUBJECT TO TSCA."	 Fill in date. Check "Negation". 	
COMPANY NAME: Your Hospital/Clinic Name COMPANY ADDRESS: Hospital/Clinic Address AUTHORIZED NAME: Healthcare provider's name AUTHORIZED SIGNATURE: Healthcare provider's signati TITLE: Healthcare provider's title FEDERAL EXPRESS AWB#: 12-digit waybill number RETURN TO: PreventionGenetics	 Complete field as indicated. Write in the FedEx 12-digit AWB Waybill Numb Tracking Number 	/ oer /

INTERNATIONAL SHIPPING INSTRUCTIONS SP051 - INOZYME PHARMA

PREVENTION GENETICS

PREVENTION GENETICS 3800 South Business Park Avenue Marshfield, Wisconsin 54449 Phone: (715) 387-0484 • www.PreventionGenetics.com

INTERNATIONAL SHIPPING INSTRUCTONS

Follow the instructions below and include the completed forms with the package to ensure the specimen can be shipped internationally to PreventionGenetics without delay.

The following two documents must accompany every sample:

- Commercial Invoice (three copies)
- Toxic Substance Control Act (TSCA) Certification Form

Complete the following steps when returning samples:

- Place sample, completed Test Requisition form and signed Informed Consent form inside Kit Box.
- Insert the Kit Box into the Federal Express (FedEx) Lab Pak with Pre-Paid label already on it and seal the Pak.
- □ Locate the Commercial Invoice and Toxic Substance Control Act Form (TSCA) in the clear pouch on the outside of the FedEx Lab Pak.
- Check for a signature on all three copies of the Commercial Invoice. If no signature is present, please sign all three copies (see example).
- □ Complete TSCA form. See instructions on the back of this brochure.
- □ Place the 3 copies of the Commercial Invoice and TSC for into the clear pouch with TSCA form on top.
- □ The sample is ready to ship. Give to FedEx Driver or call FedEx for pick-up.

Contact PreventionGenetics with questions at:

(715) 387-0484 or clinicaldnatesting@preventiongenetics.com

Invoice Number Export Dt 1/17/2017 Shipper:	Purpose of Shipmant C.I. References Clinical Testing Xic	Commercial Invoice Curr BSD PRgs 1 BOX Consignee:	ACI-71809471780 Ult Dest. CA Bill T/C 2359-5630-5	
roker dEx Ground's B: rt Nbr: Code: Sc: Clinical Te	roker	Varia (m.	Contry MTG-105 Net Wytz 3.0	0 lbs
0.5	solidecion kit for non-i	t Qty: 1.00 EA Unit Value: Infectious exempt biological substa	nce for diagnostic purpos	
mporary Export (Other Clinical Testing Kit			
ese commodities, lited States in a rersion contrary ments:	ther Clinical Testing Kit technology, or software were coordance with the Export Adm to United States law is proh	Total Shipment Weight: exported from the	1 Total Commodity Value: Terms of Sale: Fraight: Insurance: Other: Total Invoice Value:	FCA 0.00 0.00
ese commodities, ited States in a version contrary mments: cking Numbers:	Ther Clinical Testing Kit technology, or software were coordance with the Export Adm to United States law is proh	Total Shipment Weight: exported from the inistration Regulations. Bhited.	1 Total Commodity Value: Terms of Sale: Treight: Insurance: Other:	FCA 0.00 0.00

Commercial Invoice Example